


TENA Comfort

Breathability for healthy skin

New TENA Comfort with ConfioAir is an incontinence pad designed to maintain skin health and promote comfortable protection.

The new fully breathable ConfioAir back-sheet allows skin to breathe and maintain dryness for comfort and healthy skin.

The unique bowl shape provides leakage security and close body fit and is designed for use with TENA fixation pants. TENA Comfort with ConfioAir promotes comfort and dignity.

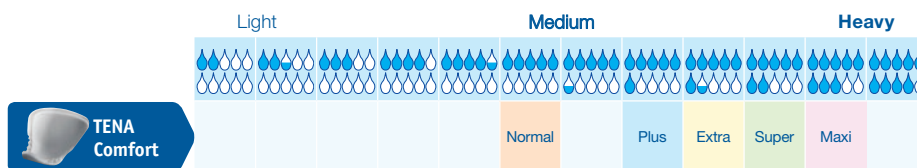

Product	Article Number	Working Absorbency	Total Packaging
TENA Comfort Normal	752742	450mls	126 (3 x 42)
TENA Comfort Plus	752846	650mls	92 (2 x 46)
TENA Comfort Extra	753040	800mls	80 (2 x 40)
TENA Comfort Super	758136	950mls	72 (2 x 36)
TENA Comfort Maxi	759128	1300mls	56 (2 x 28)

For the User: A comfortable, discreet and secure solution which keeps the skin dry.

For the Carer: Easy to fit and change. Colour-coded for correct product selection.

For the Purchaser: Less frequent changing and a greater flexibility in choice of protection level help to control and reduce costs.

Feature


Benefit

- ConfioAir allows skin to breathe.....Breathable back-sheet maintains dryness for comfort and healthy skin.
- Unique bowl shape.....Provides leakage security and close body fit.
- Designed for use with TENA fixation pantsPromotes comfort and dignity.
- Dermatologically tested.....Fast acting, highly absorbent core
- Secures leakage protection.....For comfort and dignity.
- Feeldry™ LayerFor quick absorption of liquids for maximum leakage security; enhanced dryness for improved skin care
- Odour Neutralizer.....To reduce any unpleasant smells
- Wetness IndicatorTo advise when the pad needs changing
- Quick and easy identificationColour coded absorbency level, clear size and product type indicators on product and pack


TENA Comfort Fitting Guide – Standing and Lying




Fold the pad in half lengthways before opening it out to secure a bowl shape. The front of the pad is the smaller area. Place larger area at the back.

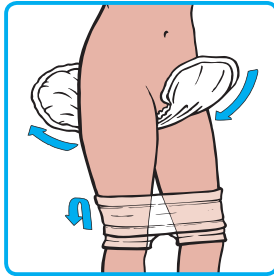


Pull fixation pants to mid-thigh then turn waistband down to the knees. Pass pad from front to back.

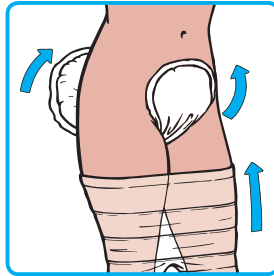


Pull pad up into position. The wetness indicators are the blue lines across the pad.

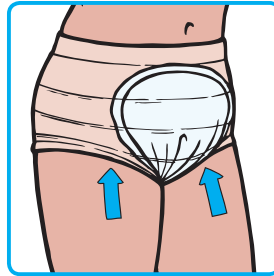

Ease leg elastics into the groin area for secure fixation and leakage security.

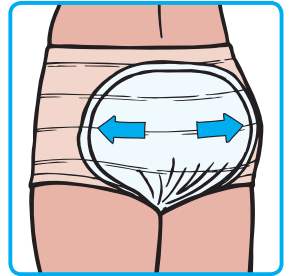


Fan-out the product to the sides (front and back) to ensure a close-body and comfortable fit.

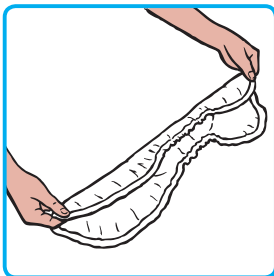


Fold the pad in half lengthways before opening it out to fit to secure a bowl shape. The front of the pad is the smaller area. Place larger area at the back.




Pass pad from front to back. The wetness indicators are the blue lines across the pad.

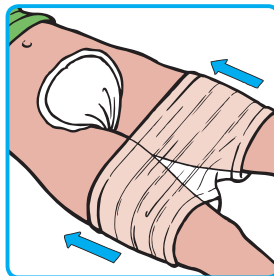

Pull fixation pants up over the pad.

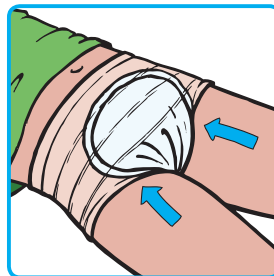

Ease leg elastics into the groin area for secure fixation and leakage security.

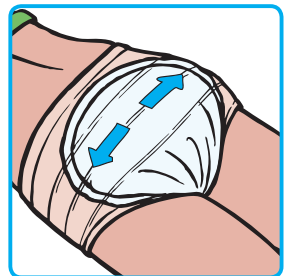

Fan-out the product to the sides (front and back) to ensure a close-body and comfortable fit.

TENA Comfort dos and don'ts

- ✓ Assess the client for correct absorbency
- ✓ Measure client around the waist for correct fixation pant size
- ✓ Always fold pad lengthways to create bowl shape
- ✓ Insert pad from front to back
- ✓ Ensure pad is fitting close to the body with wider part to the back
- ✓ Fixation pants should always be used with Comfort range
- ✗ Never place one pad inside another to increase absorbency
- ✗ Never use talc as it reduces absorbency of the product
- ✗ When creams are prescribed, use sparingly to ensure the product works effectively
- ✗ The products are designed to be body worn, do not use as bed or chair protection

People who bought TENA Comfort also bought...



TENA U-test

TENA U-test offers a solution for easy and comfortable detection of urinary tract infections. Individuals are spared unnecessary discomfort and maintain dignity, while lowering the workload for carers.



TENA Shampoo Cap

The new TENA Shampoo Cap is pre-moistened with a non-rinse hair washing formula. Ideal in situations where normal hair washing is challenging or not possible.



TENA Wet Wash Glove

TENA Wet Wash Glove enables a comfortable and convenient cleansing procedure. Impregnated with a non-rinse cleansing lotion.

SCA Hygiene Support



Email: hcmarketing@sca.com
www.TENA.co.uk



TENA Education Academy



TENA Nursing Support



SCA Care of Life. Because our products make life easier for you and for millions of people around the world. Because our resources and the way we work are natural parts of the global lifecycle. And because we care.

